


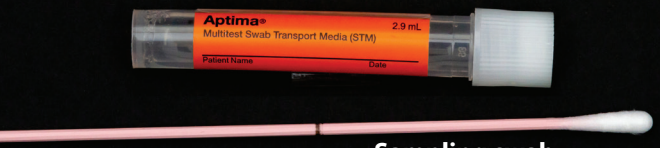

Microbiology Collection and Transport Systems


BACTERIA, MYCOBACTERIA, AND FUNGI

	<p>BD™ ESwab™ Regular Collection Kit White cap w/Liquid Amies and a regular flocked swab For all aerobic, anaerobic, acid fast bacilli, and fungal culturing <i>Lawson # 92337</i></p>
	<p>BD™ ESwab™ Minitip Collection Kit Green cap w/Liquid Amies and a minitip flocked swab For culturing as above and Pertussis PCR <i>Lawson #92339</i></p>
	<p>BD™ FecalSwab™ Stool Pathogen Collection Kit Orange cap w/liquid Cary-Blair medium and a regular flocked swab. Replaces black capped, gel Cary-Blair medium. For culturing of stool pathogens only. Distributed to patients by laboratory</p>

STD PCR GC/CHLAMYDIA/ TRICHOMONAS

 <p style="text-align: center;">Sampling swab</p>	<p>Vaginal Swab Orange Label with pink swab Gonorrhoea, chlymydia, and trichomonas testing Note: this device may also be used for rectal, eye, and throat referral GC/CT testing.</p>
 <p style="text-align: center;">Cleaning swab Sampling swab</p>	<p>Male Urethral or Female Endocervical Swab White swab is for cleaning sample site only. Note: this device may also be used for rectal, eye, and throat referral GC/CT testing Note: Trichomonas testing can not be performed off of male urethral swabs, please collect first catch urine (see below)!</p>
	<p>Male/Female 'Dirty Catch' Urine Patient collected first catch urine for all STD</p>

VIRAL PCR TESTING TRANSPORT

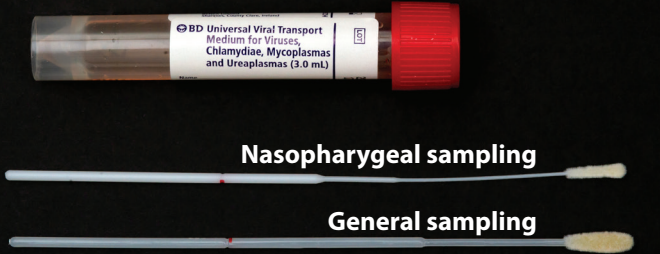 <p style="text-align: center;">Nasopharygeal sampling General sampling</p>	<p>Universal Viral Transport 3 mL Collection Kit Red cap with a regular flocked swab and a flexible minitip flocked swab For all viral testing (rapid antigen or PCR), mycoplasma/ureaplasma PCR, and chlamydia culturing <i>Lawson # 92340</i></p>
---	--

Please contact Sarah Stoner in Microbiology ext. 53138 with questions.