

CT Trauma Thorax

Application Examples: Rib fractures	
Scan Protocol (See for Technical Factors)	CT CAP or CT Chest

Reformation: 3D VRs

1. *3D VR with scapulae*. Load thin data set to GL Bony workflow. Using Batch mode, rotate the volume from AP projection to the left 360 degree with image number set at 36. Capture again to close rotation.

Series Description: **3D**

Series Number: **1001**

2. *3D VR without scapulae*. Remove any extra bony anatomy such as the chin and scapulae. Using Batch mode, rotate the volume from the AP projection to the left 360 degree with image number set at 36. Capture again to close rotation.

Series Description: **3D**

Series Number: **1002**