


CT Sacrum

Application Examples: fracture	
Scan Protocol (See for Technical Factors)	Sacrum

Reformations: Coronal, Sagittal and Oblique Axial MPRs and 3D VR.

1. *2x2 mm Coronal and Sagittal MPRs.* Align all three view ports in true orthogonal planes to sacrum before creating MPRs as depicted below.

Series Description: **COR**
SAG
AXIAL OBL

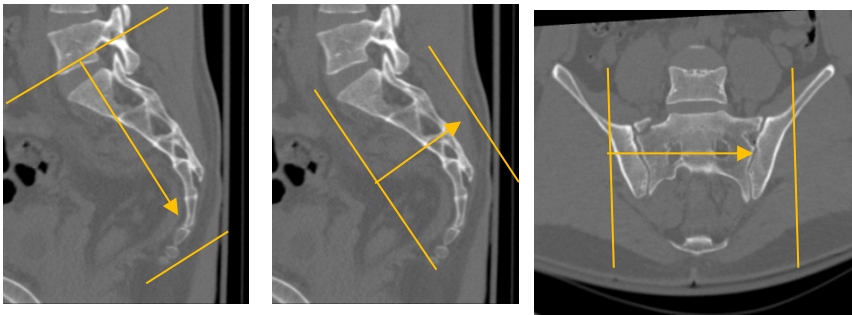

2. *3D VR rotating 360 degrees* with image number set at 36. Capture one last image to complete rotation.

Series Description: **3D**
Series Number: **1001**

