

## Pediatric Head for Trauma with 3D

Siemens go.All

Application Examples: trauma	
Scan Protocol (See for Technical Factors)	<b>Peds Head 0-18m</b>

**Reformations:** 3D VR of cranium.

1. *3D VR of cranium.* Tilt the head down as depicted below to view fontanel. Rotate to left 360 degrees with image number set at 36. Capture one last time to close rotation.

Series Description: **3D**

Series Number: **1001**



2. *3D VR of cranium.* Tumble head down and rotate 360 degrees with image number set at 36. Capture one last time to close rotation.

Series Description: **3D**

Series Number: **1002**

