


Dementia / Memory Loss Head

Application Examples: memory loss

Sequence	FoV	Thickness	Gap
SAGITTAL SPGR NeuroQuant -send to age and tba RECON AXIAL AND SAGITTAL ONLY	25.6cm	1.2mm	No gap
AX T2 TSE PROPELLER	24cm	4mm	1mm
AX T2 FLAIR PROPELLER	24cm	4mm	1mm
AX SWAN	24cm	4mm	1mm
AX DWI (B-0, B-1000)	24cm	3mm	1mm
SAG CUBE FLAIR (RECON AX & COR 1 x1)	24cm	1.2mm	NO GAP
IF ORDERED WITH GAD DO ENHANCED SEQUENCES			
INJECT CONTRAST (1mmol/Kg)			
SAG T1 FS SPGR WITH GAD	24cm	4mm	1mm
AX T1 FS SPGR WITH GAD	24cm	4mm	1mm

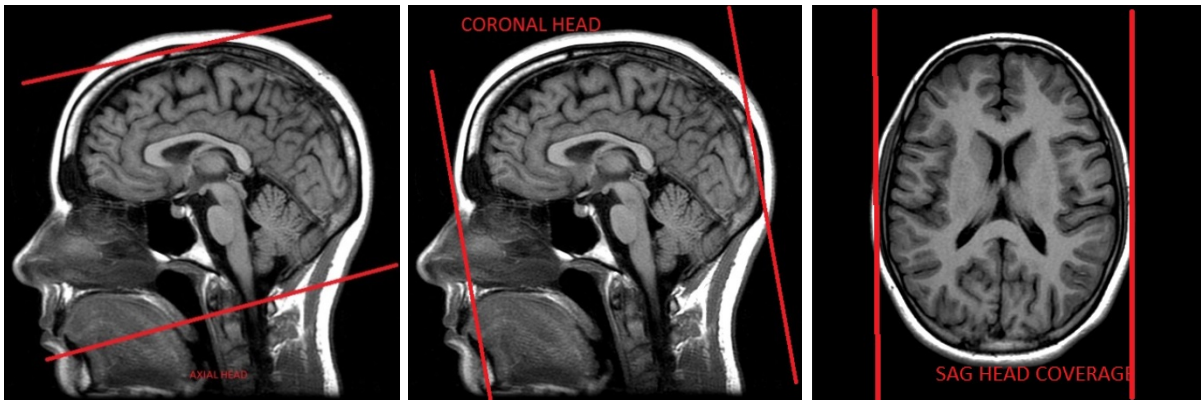

AXIAL Coverage

COR Coverage

SAG Coverage