


# ADULT SEIZURE HEAD


Application Examples: seizure activity

Sequence	FoV	Thickness	Gap
AX T2 TSE PROPELLER	24CM	4MM	1MM
AX T2 FLAIR PROPELLER	24CM	4MM	1MM
AX SWAN	24CM	4MM	1MM
COR HIRES T2 ( <i>THRU TEMPORAL LOBE ONLY</i> )	20MM	3MM	1MM
AX DWI (B-0, B-1000)	24CM	3MM	1MM
Sagittal SPGR 3D NeuroQuant – RECON AX/SAG ONLY send to neuroquant_asym	25.6cm	1.2mm	No gap
INJECT CONTRAST (1mmol/Kg)			
COR T2 FLAIR WITH GAD ( <i>THRU TEMPORAL LOBE ONLY</i> )	24MM	3MM	1MM
SAG CUBE FLAIR WITH GAD ( <i>RECON AX &amp; COR 1 x1</i> )	24CM	1.2MM	NO GAP
SAG T1 FS SPGR WITH GAD	24CM	4MM	1MM
AX T1 FS SPGR WITH GAD	24CM	4MM	1MM


TEMPORAL LOBE COVERAGE

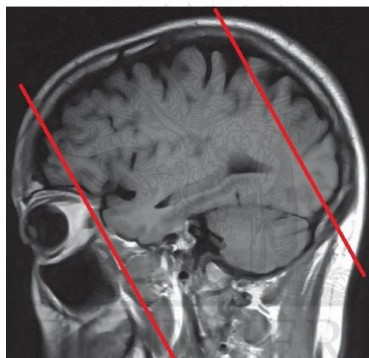

COR HIRES T2 TEMPORAL LOBE COVERAGE

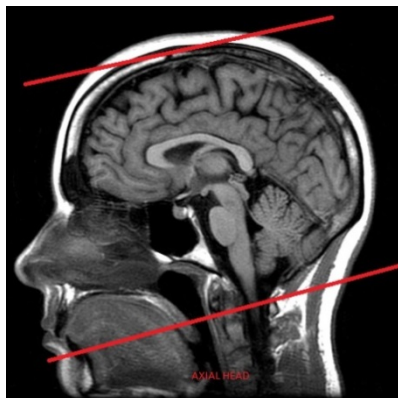

AXIAL Coverage

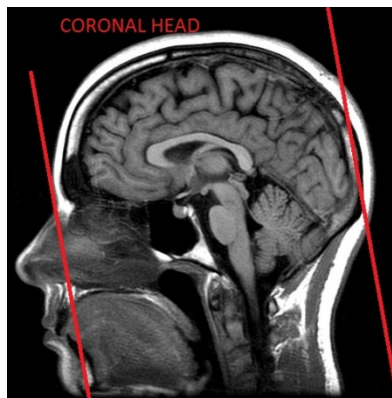

COR Coverage



SAG Coverage