


Lumbar Spine Post Op

Application Examples: post-operative lumbar

Sequence	FoV	Thickness	Gap
SAG T1	28CM	4MM	1MM
SAG T2	28CM	4MM	1MM
SAG STIR	28CM	4MM	1MM
AX T2 L3-S1	17CM	4MM	1MM
AX T1 L3-S1	17CM	4MM	1MM
AX DWI (VALUES B-0,B-1000)	17CM	4MM	1MM
INJECT CONTRAST (1mmol/Kg)			
SAG T1 FS WITH GAD	28CM	4MM	1MM
AX T1 FS WITH GAD	17CM	4MM	1MM

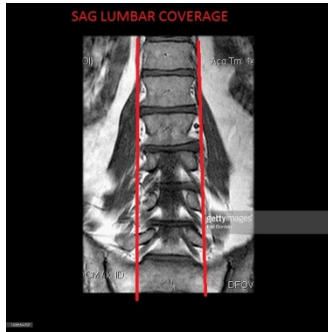

SAG Coverage



AXIAL Coverage